


INSTRUCTIONS FOR USE

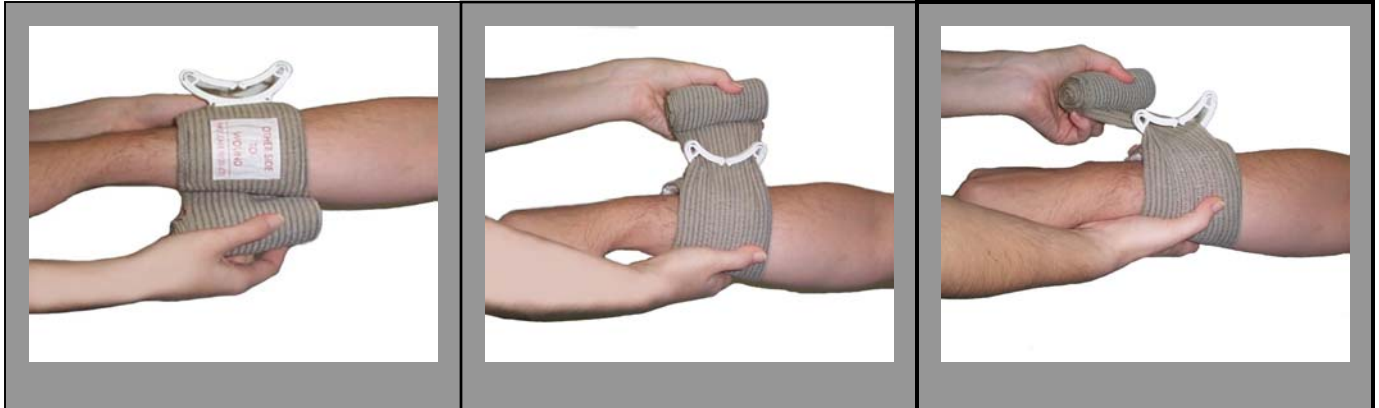

1 Place pad on wound & wrap the elastic bandage around limb or body part

2 Insert elastic bandage into pressure bar

3 Tighten elastic bandage



4 Pull back – forcing pressure bar down onto pad

5 Wrap elastic bandage tightly over pressure bar and wrap over all edges of pad

6 Secure hooking ends of closure bar into elastic bandage

THE EMERGENCY BANDAGE

FIRST CARE PRODUCTS

3050 Post Oak Blvd., Suite 1710

HOUSTON, TEXAS 77056

Tel: (713) 723-6000

Fax: (713) 723-6221

Web: www.firstcareproducts.com

Email: info@firstcareproducts.com